

## CLEANING

When cleaning the grip ring product, it is necessary to use a safe and sanitary product.

Suggestions: Asepti-Wipes, a low alcohol cleaner, or with warm water and soap.

Any other product you might consider **should not** include any harsh ingredients such as: acetone, bleach, xylene, or monochloro toluene. Most products that are safe to use on plastics are usually safe to use on these products.

## DIRECTIONS

Any cleaning/sanitation product chosen to use should be used gently. **Do not** scrub or soak with any solution.

At the first sign of stickiness and/or discoloration of the grip ring, discontinue that cleaning/sanitation product.

## GUARANTEE

Grip rings are constructed of hardy closed-cell foam and coated with a vinyl-like material. When taken care of properly, the rings will give excellent service.

Because patients vary in hand and feet size, stretching the rings may cause small breaks on the inner side of the rings.

We only guarantee this product for a period of 90 days.

**End of Document**